Bruxism “S” Splint

DIRECTIONS FOR USE

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MODE D'EMPLOI

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Manufacturer:

Dentsply International Raintree Essix

7290 26th Court East

Sarasota, FL 34243 USA

Tel: 800.883.8733

www.essix.com

R_x
ONLY

SINGLE
USE ONLY

Bruxism “S” Splint

ENG

Indications for Use:

Indicated to protect teeth from abrasion due to bruxism.

Contraindications: None known

⚠ **Warnings:** None known

Precautions: None known

Adverse Reactions: None known

Step By Step Instructions: The Bruxism “S” Splint attaches directly to fixed orthodontic appliances. This full-coverage, flat-plane occlusal splint (Fig.1) allows tooth movement to continue while preventing excessive tooth wear. It can be used on either the Maxillary Arch (Fig.2) or Mandibular (Fig.3) Arch.

Fig.1 Unmodified Bruxism “S” Splint.

Fig.2 Splint fitted to Maxillary Arch.

Fig.3 Splint fitted to Mandibular Arch.

The Bruxism “S” Splint does not require any impressions or lab work and is easily adjusted by dipping it in hot water (180°F/82.2°C) and bending the clips (Fig. 4) to conform to the patient’s fixed appliance (Fig. 5).

Fig.4 Clips bent inward with pinching motion.

Fig.5 Clips adjusted to fit over the fixed appliance.

The distal ends, inner flanges and retentive clip edges of the splint can be trimmed with scissors for further patient comfort and can be modified with an acrylic bur (Fig.6 and Fig.7).

Fig.6 The splint being modified.

Fig.7 The splint being trimmed with scissors.

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