USER MANUAL

Precision Electric Heat Knife

Items #88550 and #88550-240

Precision Electric Heat Knife

BASIC OPERATING INSTRUCTIONS

1. Heat knife must be operated from a grounded outlet.
2. Clean fittings with new blades are recommended.
3. When cutting a 1mm plastic material, adjust the dial to 5 for optimum cutting without overheating.
4. When cutting materials of other thicknesses, establish the setting for optimum cutting by experiment.

CLEANING AND MAINTENANCE

Cleaning Blade During Use

Turn heat up to 9 for approximately 10 seconds; then re-adjust to cutting setting. Wipe the knife tip clean on moistened sponge.

Cleaning Brass Mounting

During normal use, molten plastic can flow between the blade and its brass mounting, particularly if a fitting is loose. This will stop the heat knife from working and should be cleaned promptly.

Replacing Blade

Before replacing blade, unplug the heat knife from the outlet and ensure the old blade is cool. Scrape away any melted plastic residue from the flat surface of each brass blade mounting. Fit the new blade.

TROUBLESHOOTING

1. **Orange indicator light does not illuminate:** Check the fuses on the back of the unit and on the plug.
2. **Orange indicator light is illuminated, but heat knife does not function** - check the following:
 - a. Blade may be cracked and will not heat. Blade needs to be replaced.
 - b. Melted plastic residue between the blade and its brass mounting may act as an insulator causing the heat knife not to operate. The brass mounting needs to be cleaned.

WARNING:

HEAT KNIFE BECOMES HOT DURING USE! When trimming plastic, always cut away from yourself to reduce the risk of injury.

For continued protection against risk of fire, replace only with same type and rating of fuse. Disconnect power before changing fuse.

WARRANTY TERMS AND CONDITIONS

The Precision Electric Heat Knife is warranted for its designated use, in accordance with the instructions, against original defects in material and workmanship for a period of 3 months* from date of purchase. This warranty extends only to the original purchaser only.

*Warranty void if unit is tampered with or if there is evidence of tampering.

Blades, sponges and fuses are excluded from warranted parts.

REPLACEMENT PARTS

- (1) Blade and fitting screws#88550-BLADE
- (1) Fuse#88550-FUSE
- (1) Sponge.....#88550-PADS

REPAIRS AND REPLACEMENT PARTS

Contact Customer Service:

Dentsply International Raintree Essix
 7290 26th Court East
 Sarasota, Florida 34243 USA
 Phone: (800) 883-8733

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