MTM[®] Clear•Aligner In-Office Use & Care

 Manufacturer:
Dentsply International Raintree Essix
7290 26th Court East
Sarasota, FL 34243 USA
Tel: 888.898.4mtm (4686)

MTM® Clear•Aligner In-Office Use & Care

ENG

For Professional Use Only – Rx Only

NOTE: Please provide a copy of these use and care instructions to each MTM® Clear•Aligner In-Office In-Office Patient.

Device Description:

MTM® Clear•Aligner In-Office consists of a series of custom made removable plastic orthodontic appliances that sequentially position teeth by way of continuous gentle force.

Indications for Use:

MTM® Clear•Aligner In-Office is indicated for the treatment of anterior tooth malocclusions in patients with permanent dentition (i.e. all second molars). Utilizing a series of incremental minor tooth movements movements (torque, tipping, rotation and bodily movement), MTM® Clear•Aligner In-Office sequentially positions teeth by way of continuous gentle force.

Contraindications:

Only for patients with permanent dentition.

⚠ Warnings:

None known.

Precautions:

Patients with a history of allergic reactions to plastics should not use this product.

Adverse Reactions:

Allergic reactions, to plastics.

Wearing Your Aligner(s):

Aligners should be worn at all times (day and night), except while eating, drinking or brushing your teeth. Once you have finished eating or drinking, follow proper oral hygiene and place the aligner(s) back into your mouth immediately. Failing to comply with wear requirements can adversely affect tooth movement and may add to both your treatment time and/or expense.

NOTE: Drinking hot liquids (coffee, tea, etc.) while wearing your aligners can distort the aligner, thus compromising your treatment outcome.

Oral Hygiene:

During treatment it is important to maintain good oral hygiene. Remove your aligner(s) before eating or drinking. Immediately after each meal, brush and floss your teeth prior to reinserting your aligners. Trapping food and certain liquids in your aligner(s) during treatment can lead to decay.

Store Your Aligner(s) Securely:

When your aligner is not in your mouth, store it in the protective case provided to you by your dental professional. Leaving aligners loose or lying around makes it easy for them to be accidentally lost or damaged. Lost or damaged aligners may result in an additional replacement fee and can lengthen treatment time. Also, do not store aligners near a heat source, as this may cause them to distort, resulting in improper fit in your mouth.

Chewing Gum:

Do not chew gum while wearing your aligner. The gum can stick to the surface of the aligner.

Cleaning your Aligner(s):

We recommend you clean your aligner once a day or as necessary. Don't use toothpaste, as this can scratch and dull the plastic, adversely affecting the aligner's clarity and possibly resulting in breakage. Retainer Brite® Cleaning Tablets are recommended to clean your aligner safely and leave it fresh. Follow the directions provided with Retainer Brite® Cleaning Tablets to clean your aligner. Ask your dental professional for Retainer Brite® Cleaning Tablets, or visit www.Smileshop.com.

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