# TRAYLUX™

## Light Curing Unit

MODEL M5

Distributor:

**Dentsply International Raintree Essix**

7290 26th Court East

Sarasota, FL 34243 USA

Tel: 800.883.8733

[www.essix.com](http://www.essix.com)

#MKLCB RTE-252-17 Rev.3 10/17

# TRAYLUX™ Light Curing Unit

ENG

Model M5

## Step-By-Step Instructions:

1. Unpack light curing unit from packaging. Remove foam shipping insert and power cord from inside drawer. Remove protective paper from drawer door.
2. Plug slotted end of power cord into back of unit. Plug pronged end of power cord into grounded electrical outlet.
3. Turn power “On” using toggle switch on back of unit. Timer display will light up.
4. Curing time can be adjusted between 6 seconds (0.1) and 99 minutes (99). Use the “Up” or “Down” arrow buttons on top of unit to select desired curing time.
5. Open drawer.
6. Place appliance to be cured into tray.
7. Close drawer.
8. Press “Start/Stop” button located on top of unit to begin curing. Timer will count down the selected time.
9. Curing can be interrupted by opening the drawer. The light will shut off automatically.
10. Once the curing cycle is complete, open the drawer to remove the cured appliance.
11. The last programmed time will remain in memory until the “Up” or “Down” arrow buttons are used to change it.

TRAYLUX is not a trademark of Dentsply Sirona.

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