USER MANUAL

Turbomax[®] Alginate Automatic Mixer

For #AM001 and #AM002

THANK YOU FOR PURCHASING THE TURBOMAX® ALGINATE AUTO MIXER.

Prior to using this product, please read this User Manual carefully for set-up, usage and maintenance instructions. Please keep this User Manual handy as a reference for your Auto Mixer.

ENG

NOTES TO CUSTOMERS:

1. Dentsply International Raintree Essix shall not assume any responsibility for any malfunction, damage, or accident of personal bodily injury due to the improper removal, modification, maintenance, or repair of the Turbomax® Alginate Auto Mixer by parties other than us.
2. Dentsply International Raintree Essix shall not assume any responsibility for any malfunction, damage, or accident of personal bodily injury due to using a product not provided by us.
3. Dentsply International Raintree Essix shall not assume any responsibility for any malfunction, damage, or accident of personal bodily injury due to the improper removal, modification, maintenance, or repair of the Turbomax® Alginate Auto Mixer that did not include parts provided by us.
4. Dentsply International Raintree Essix shall not assume any responsibility for any malfunction, damage, or accident of personal bodily injury due to the improper operation of the Turbomax® Alginate Auto Mixer not in compliance with this User Manual.
5. Dentsply International Raintree Essix shall not assume any responsibility for any malfunction, damage, or accident of personal bodily injury is caused due to the usage environment not being in compliance with the power, installation environment and conditions specified in this User Manual.
6. Dentsply International Raintree Essix shall not assume any responsibility for any malfunction, damage, or accident of personal bodily injury due to fire, earthquake, flood, or lightning.

PREVENTING ACCIDENTS AND MALFUNCTION

To avoid any malfunction of the machine or personal bodily injury, the operation of the Turbomax® Alginate Auto Mixer shall be used in compliance with Important Notes and Notes for the Use of Medical Therapeutic Machine (Safety and Danger Prevention) specified in this User Manual.

- DANGER** - It is possible to cause death or serious injury
- WARNING** - It is possible to cause damage to goods
- NOTES** - It is possible to cause light personal injury or damage to goods
- IMPORTANT** - Please review the Important Notes in order to avoid a malfunction of the machine.

IMPORTANT INFORMATION

Use of the Turbomax® Alginate Auto Mixer is intended for dental professionals.

PARTS AND ACCESSORIES

1. Main power switch
2. Power button
3. Start/Stop button
4. Memory selection button
5. Mixing time indicator
6. Adjustment buttons for mixing time
7. Display of memory position
8. Power cable
9. Power cable receptacle
10. Fuse
11. Mixing cups #AMCUP-R
12. Machine body
13. Spatula for mixing cup #AMSPA
14. Stirrer stick #AMSTR
15. Water Dosing Bottle #AWDB01
16. Magnetic accessory basket #AMBAS

IMPORTANT NOTES REGARDING USE

The following items should be closely noted to assure the correct operation of the Turbomax® Alginate Auto Mixer, free from malfunction or damage to the machine, and/or personal injury:

1. To extend the setting time of the alginate powder, please use cool water (50° - 68° F / 10° - 20° C) for mixing. If using fast-setting alginate powder, watch the setting time.
2. The weight limit of the mixing amount is 25 - 100 g, never to exceed 100 g.
3. If the small hole at the center of mixing cup cap is clogged or blocked, it will be difficult to tighten the cap completely. The blockage should be cleared immediately.
4. If there is any abnormal noise or vibration while the machine is mixing, press the STOP button immediately and lift the machine cover to check mixing cup placement.
5. The mixing cup is an accessory item and is not covered by the machine warranty. If a mixing cup cannot be tightened completely, please switch to a new cup immediately.
6. Please retain the Mixer's original packing materials (boxes, foam inserts). If the machine is to be sent for repair or transported to another location, please fasten the motor by using the two fixing bolts attached for handling (Fig. 1) and place the mixer in its original packaging. NOTE: Not securing the motor with the two fixing bolts when transporting voids the product warranty.

Figure 1

INSTALLATION AND USE

1. CONDITIONS FOR INSTALLATION AND USE

- For indoor use only
- Please use at temperatures between 41° - 104° F / 5° C - 40° C
- Please use the appropriate power voltage for your mixer AM001 (AC 110V), AM002 (220V).

2. INSTALLATION

- Install on a stable counter or table with a flat surface
- Never install near flammable substances

IMPORTANT INFORMATION

3. FASTENING BOLTS FOR TRANSPORTATION

- Prior to installation, please remove the two fastening bolts fixed at the bottom of machine (Fig 1). If the mixer is to be sent for repair or brought to another location, please fasten these fixing bolts to avoid malfunction or damage. Not securing the motor with the two fixing bolts when transporting voids the product warranty. Please store the two fastening bolts properly for future use.

4. POWER

- Connect the power cable to the outlet on the reverse side of machine (Fig. 2 - indicated by the circle).
- Plug 3-prong, grounded end into power source (wall receptacle).

MIXING METHOD

1. MIXING OF ALGINATE POWDER

- Turn mixer on at main power switch on the reverse side of machine (Fig. 3)
- Push the power button located on the control panel - digital display on the operation panel will come on (Fig. 4)
- Prepare the alginate powder and cool water (50° - 68° F / 10° - 20° C)

Figure 2

IMPORTANT INFORMATION

The hardening time of the alginate can be controlled by water temperature. If cool water is not used the hardening time will be shortened.

- Measurement of the alginate powder should be done according to the alginate manufacturer's instructions. Add the corresponding amount of water. (Fig. 5A & 5B)
- Using the Stirrer stick, stir the alginate powder and water together for 2 seconds before capping the mixing cup and loading it into the machine. (Fig. 6)
- Close the mixing cup by holding the cap and turning it as indicated on the mixing cap. (Fig. 7)

IMPORTANT INFORMATION

The mixing cup cap must be completely tightened or the alginate powder will splash during mixing and cause the machine to malfunction. The mixing cup and mixing cup cap are considered accessory products and not included in the machine warranty. If the cap cannot be secured tightly, please use a new mixing cup and cap.

- Place the mixing cup inside the mixing well (Fig. 8)
- Close the cover of mixer. The mixer will not start if the cover is not secured properly.
- Based upon the amount of alginate powder added to the mixing bowl, set the timer (please refer to the "Setting for Mixing Time"). Once the timer is set, press the start button (Fig. 9). The digital display will count second by second and will display the time of completion. The cover of mixer is fully locked when running and cannot be opened.

IMPORTANT INFORMATION

Should you need to stop the mixer during mixing, press the STOP button and the mixer will immediately stop. During mixing, should the mixer detect any abnormal condition, it will stop automatically.

- After making sure that the mixer has stopped, open the cover and remove the mixing cup.
- Use the spatula to remove the alginate material. (Fig. 10)
- Alginate material will set bubble free. (Fig. 11)

Figure 3

Figure 4

Figure 5A

Figure 5B

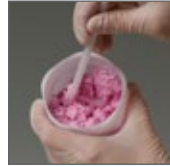
Figure 6

Figure 7

Figure 8

Figure 9

Figure 10

Figure 11

MIXING TIMES

1. SETTING THE MIXING TIMER

- The timer has 3 pre-programmed settings:

Memory	Mixing Time (sec.)
1	8
2	10
3	12

Figure 12

Figure 13

- Press the MEMORY SELECT button to select the time required for mixing.
- To alter or change the timer pre-sets temporarily, press the UP and DOWN buttons simultaneously and key in the desired mixing time. The maximum mixing time is 16 seconds (1 second is needed to process the requested time adjustment). When the mixing is completed, the original pre-set time will return. To permanently change the timer pre-sets, press the UP and DOWN buttons simultaneously - the mixer will sound 2 electronic beeps and the timer display panel will flash twice. In order to store the new time in memory, turn the power switch off and then back on to confirm that the new time is stored.
- A grid of mixing times, time settings (water temperature between 50°-68°F / 10°-20°C) are listed below:

Alginate Scoops	Mixing Time (sec.)	Impressions Made
1	8	Small Full Arch
2	10	Medium Full Arch
3	12	Large Full Arch

IMPORTANT INFORMATION

The mixing times listed above are average times. If there is any difference between water temperature and room temperature established by the supplier of alginate impression powder, then the times will be different.

MAINTENANCE

1. MIXING CUP

- After setting, the remaining alginate material inside the mixing cup can easily be removed. The small hole at the center of cap should be cleaned to ensure that it is not clogged or blocked in any way. (Fig. 12)

IMPORTANT INFORMATION

- The small hole at the center of cap must remain clear. If it becomes clogged or blocked, then the mixing cup cap will be difficult to open or close.
- Any water remaining after cleaning must be removed completely, otherwise it will cause a mixing error.

2. CLEANING THE MIXER

- If there is any dirt outside on the mixer, please use a cloth to clean. Use a neutral cleaning agent to remove large amounts of debris (Fig. 13).

- **DO NOT CLEAN THE INSIDE OF THE MIXER/MIXER WELL WITH WATER.**

IMPORTANT INFORMATION

Never use solvent or volatile chemicals for cleaning. This will cause the machine surface to fade.

TROUBLESHOOTING IMPORTANT INFORMATION

If you experience a malfunction, please try to resolve the problem by using the chart below. If the machine is still malfunctioning, please contact Dentsply International Raintree Essix at 1-800-883-8733 Monday – Friday: 8:30 – 5pm CST for further instructions and assistance.

PROBLEM	CAUSE	METHOD
Mixer does not power on/turn on when you press the power button	Bad connection of power cord	Inspect the connection of power cord
	Main power switch is in off position (rear of mixer)	Turn main power switch on
	Burnt out fuse	Inspect fuse and change if need be. If the fuse is burnt out, call Customer Service at 1-800-883-8733
The digital display does not come on	Defect of PC board	Call Customer Service at 1-800-883-8733
The initial running is slow	Defect in driving motor or belt	Call Customer Service at 1-800-883-8733
	Defect in the starting condenser	Call Customer Service at 1-800-883-8733
Abnormal noise or vibration while running	Too much Alginate in mixing cup	Try smaller amount of Alginate
	Mixer started, but there is nothing inside the mixing bowl	Place properly filled mixing cup in the unit and retry
	Damaged belt	Call Customer Service at 1-800-883-8733
	Alginate powder splashes caused by loosening of mixing-bowl cap	Check if the mixing cap on the cup is tightly closed
Motor does not start or alginate mixing uneven	Incorrect amounts of alginate and water	Refer to alginate suppliers mixing ratios
	Driving motor/triangular belt or round belt is damaged	Call Customer Service at 1-800-883-8733
	The sensor lock in the top cover is malfunction	Check if the top cover is closed tightly
"E1" appears on the mixing time indicator	The top cover is not in the closed position	Close the top cover again

SPECIFICATIONS

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Weight: 16 kgs / 35.2 lbs.

Voltage: AC 220V or AC 110V (50/60Hz)

Consumed: Maximum 400W Power

Timer: 1-16 sec. adjustable by second

Memory: 3 (initial settings – 8, 10, and 12 sec.)

Revolution: 3,400 rpm. (60Hz)

2900 rpm. (50Hz)

WARRANTY AND REPAIRS

The Turbomax® Alginate Auto Mixer is warranted for its designated use, in accordance with the instructions, against original defects in material and workmanship for a period of six months from the date of purchase. This warranty extends only to the original purchaser.

WARRANTY SCOPE: During the six-month warranty period, any defect or mechanical issue with the Turbomax® Alginate Auto Mixer will be repaired free of charge.

EXCEPTIONS TO THE WARRANTY: Repairs will be done for a fee if the following conditions apply:

1. Modification or repairs have been performed by a non-authorized dealer
2. Damage caused to the machine due to a natural disaster or fire
3. Damage caused as a result of exceeding voltage or of the machine not being installed correctly per the User Manual
4. Damage caused by an undesignated connection
5. Damage caused by improper use and not in accordance with the User Manual
6. Damage or consumption of consumable products
7. Damage caused as a result of moving machine without proper fastening of fixing bolts

REPAIRS: Call Customer Service at 1-800-883-8733.

For warranty repairs, a call tag will be issued, the machine picked up and the appropriate repairs performed. **Machine must be sent back in the original packaging with the fixing bolts in place.** For non-warranty repairs, delivery instructions will be provided when you call.

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