Window-Punching Pliers

DIRECTIONS FOR USE
INSTRUCCIONES DE USO
MODE D'EMPLOI
ISTRUZIONI PER L'USO
GEBRAUCHSANWEISUNG

Manufacturer:
Dentsply International Raintree Essix
7290 26th Court East
Sarasota, FL 34243 USA
Tel: 800.883.8733
www.essix.com

#MKWP RTE-181-18 Rev.7 9/18

R_x
ONLY

Function:

The Window-Punching Pliers are designed to modify thermoplastic aligners to enable access to teeth for placement of auxiliary attachments.

Contraindications:

The Window-Punching Pliers are not designed to place force points for purposes of tooth movement.

 **Warnings:**

Treatment range using the Window-Punching Pliers is not to exceed 1mm of movement per month per tooth. Excessive rate of movement may cause root resorption.

Precautions:

None known.

Adverse Reactions:

None known.

Step by Step Instructions:

1. Pliers must be sterilized before each use. See sterilization guidelines below.
2. Locate position of desired attachment.
3. Mark location of window on aligner plastic. Make sure the window is large enough to accommodate the attachment.
4. Select appropriate Window-Punching Plier based on the position of attachment as noted in chart below.
5. Punch a hole in aligner plastic by placing plier tip at marked location and squeezing handles together. This may be repeated to enlarge the window.
6. Make sure edge of window is smooth.

Window-Punching Pliers Purpose/Indication:

Item	Purpose/Indication	Position on Aligner
Incisal Window Pliers	Punches a hole in the aligner near the incisal edge to allow for other auxiliary attachments, including buttons and bands	At the incisal edge
Gingival Window Pliers	Punches away plastic to allow for other auxiliary attachments near the gingival edge, including buttons and bands	At the gingival edge

Reference the MTM® Clear•Aligner In-Office Quick Start Guide for detailed examples of use.

Cleaning and Sterilization Guidelines:

Clean, rinse and dry before sterilization.

Sterilize by autoclave/moist heat at 121°C / 250°F for 30 minutes (gravity displacement cycle). Minimum recommended drying time is 30 minutes.

Maintenance Suggestions:

AFTER sterilizing, lubricate **SPARINGLY ONCE A WEEK** with dental handpiece oil. One drop or light spray is sufficient; wipe off excess lubricant. The major cause of staining or discoloring is **OVERUSE** of lubricants or “sticky joint” treatments. Lubricants, waxes and “sticky joint” remedies will vaporize inside the chamber and saturate all other items placed inside.

If you are experiencing brown or yellow staining, scrub with **ISOPROPYL** alcohol and steel wool (type A extra fine).

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